



EZ Slide 3501B



*From the Leader
in Top Slide
Surgical Tables*

EZ Slide 3501B is the surgical table for the utmost versatile imaging requirements



Innovations in Surgical Imaging Flexibility for Patient Care



7.5 in. (195mm) Slide to Head



13.5 in. (335mm) Slide to Foot

The EZ Slide 3501B Top Slide Surgical Table is designed to meet O.R. Suite demands of the 21st Century, including every day reliability and performance.

EZ Slide is a universally versatile surgical table providing excellent imaging flexibility, clinical versatility and cost effectiveness. EZ Slide provides an impressive range of full body imaging and articulating table functions as required for C-arm imaging orientations, without the use of multiple support accessories or having to reposition the patient. One touch of a button does it all!

Extreme Stability with 21 in. (530mm) Top Slide, 700 lbs. (318kg) lift and 600 lbs. (272kg) articulation capacity!

Wide Ranges of Movement

Maximum Room for Feet with ergonomic, narrow base design.

Optimal Radiolucence by design. Superior table top structural support eliminates metal frame interference for radiological images.

User Friendly Operation with illuminated, one touch and extremely durable pendant control.

Versatile Accessories
 Hundreds of accessories are available for every application. EZ Slide is equipped with standard side rails to accommodate customer owned accessories.



45 in. (1140mm)
 Max.
 26 in. (660mm)
 Min.



21 in. (530mm)
 Top Slide



90° Reflex



40° Flex



Leg Up 2°



98° Leg Down



90° Back Up



40° Back Down



25° Trendelenburg



20° Reverse
 Trendelenburg





20° Lateral Tilt
 Right & Left



EZ Slide 3501B imaging area


Upper Body Imaging Coverage

- Pacemaker Insertions
- Hickman Catheters
- Broviac Insertions
- Shunts
- IVC Filters
- Lung Biopsies
- Cholangiography
- Cardiovascular
- Thoracic
- Kidney
- Gall Bladder
- Laparoscopy
- Total Hip Revisions
- Ophthalmic
- Orthopedic
- Upper Limb Arthroscopy
- Neurosurgery
- Spinal
- Implantable Defibrillation
- Endoscopy

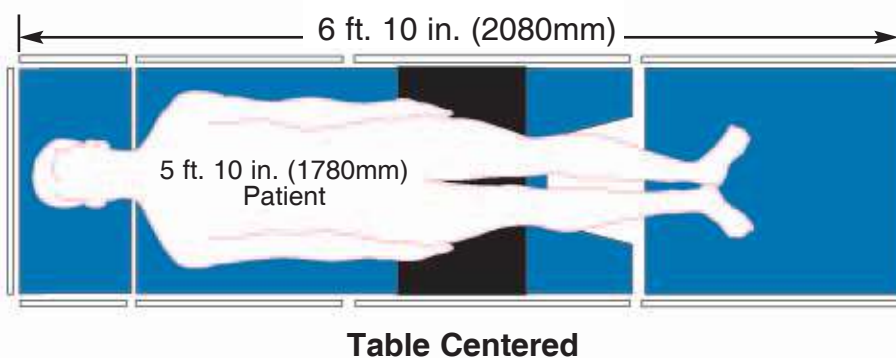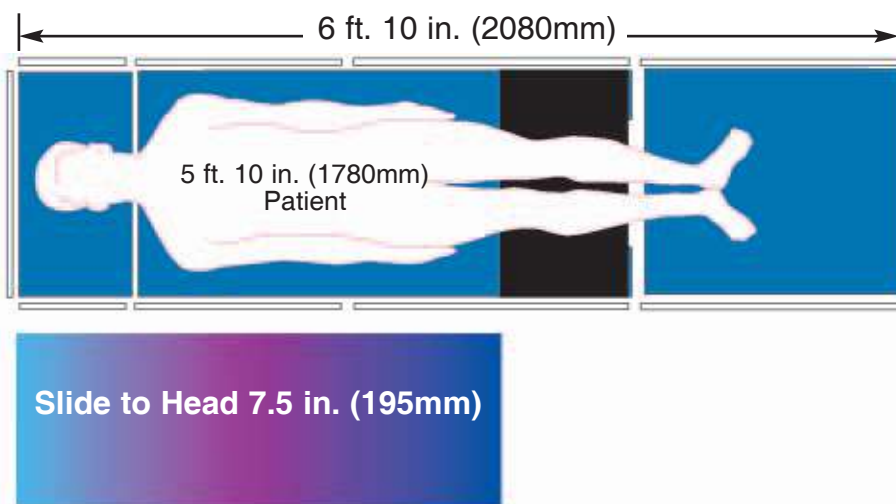

Lower Body Imaging Coverage

- Spinals
- Femoral Popliteal Bypass
- Laser Angioplasty
- Endovascular
- Orthopedics
- Lower Limb, Non-traction Orthopedics
- Laparoscopy
- Upper Limb Arthroscopy
- Cystoscopy
- Endourology
- Gynecology
- Abdominal
- Nissen Fundoplication

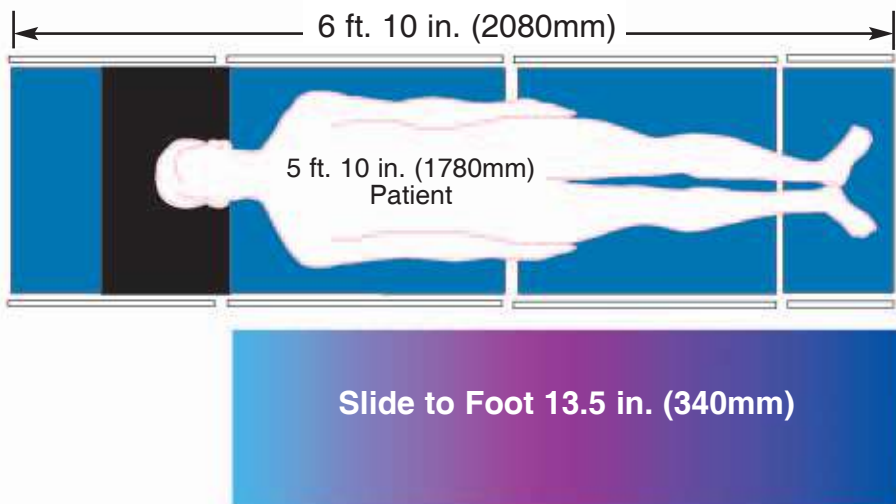

* Head section can be used as foot extension.

It's simply the easiest and most durable pendant to use from any location around the surgical table, with convenient back-lighting for high visibility when working in low light environments.

- Simple One Touch Operation
- Back-Lit Control for High Visibility
- Audible Alarm for Kidney Bridge, Back Up Articulation, Top Slide Obstruction
- Anatomically Correct Return to Level
- Top Slide Return to Center Indicator
- Recessed Buttons to Eliminate Accidental Activation
- Exceptionally Durable (including superior steel housing design), Water Resistant Pendant Control
- Can be used in conjunction with optional Three Function Foot Control



EZ Slide 3501B

optimal imaging coverage



***700 lbs. (318kg) Lift
and 600 lbs. (272kg)
Articulating Capacity***



Battery Life Indicator & Table Base,
Twist-Lock Cord Connection Prevents
Fluid Ingress.



Adjustable Head
Section with Quick
Release Bar for
Emergencies



Full Length X-Ray (Bucky)
Top Accessory for Image
Capture



Secondary Function
Switches for Backup to
Primary Control



Power Kidney Bridge
for Safe and Easy
Positioning

and top slide range



**Exclusive 21 in.
(530mm) Top Slide**



Optional 40 in. (1020mm) Carbon Fiber Extension
for Specialty Imaging Requirements



Durable, Back-lit
Pendant Control



Manual Emergency
Brake Unlock



Self Leveling Brakes for
Stability on Uneven
Floors



Unique Table
Accessories for Safe
and Improved
Positioning

EZ Slide 3501B positioning guidelines



Gall Bladder / Thoracic

Multi-Task Armboard 2-020-02
Restraint Strap 6-010-41-B
10 in. (250mm) Foot Extension /
Board 4-030-13



Vascular / Endovascular

Standard Armboard 2-010-02



Gastro / Intestinal

Raised Armboard 2-020-01-1
Sure Clamp 6-040-08
Split Leg Section 4-030-61
Split Leg Foot Rest 4-030-04

***GYN / Endourology /
Cystoscopy***

Urology Bracket Set 4-040-16
Pouches (10) 4-040-14-10
Levigator Stirrups 4-090-03-1



Kidney / Thoracic

Multi-Task Armboard 2-020-02
Chest & Waist Support (pr.)
3-010-01
Restraint Strap 6-010-41-B
Standard Armboard 2-010-02
10 in. (250mm) Foot Extension /
Board 4-030-13



Ophthalmic / ENT

Silhouetted Narrow Head Section
7-010-08
Standard Armboard 2-010-02
Restraint Strap 6-010-41-X



EZ Slide 3501B positioning guidelines



Neurosurgery

Multi-Task Armboard 2-020-02
Restraint Strap 6-010-41-B
10 in. (250mm) Foot Extension /
Board 4-030-13



Spinal

Multi-Task Armboard 2-020-02



Lumbar

Multi-Task Armboard 2-020-02
Restraint Strap 6-010-41-B
18 in. (460mm) Knee Rest
4-030-10-B

Shoulder Arthroscopy

Beach Chair Shoulder Positioner
3-040-20, 3-040-21
Raised Armboard 2-020-01-1
10 in. (250mm) Foot Extension /
Board 4-030-13



Bariatric Surgery

Table Side Extension 3-060-06 (6)
Table Pad for use with Side
Extension 1-010-36-BS



Lower Extremity / Endovascular / Podiatry

Standard Armboard 2-020-02
40 in. (1020mm) Carbon Fiber
Extension 4-030-40



EZ Slide 3501B

surgical table specifications

Technical Specifications:

Length w /o head rest	69 in.	1750mm
Length with head rest	82 in.	2080mm
Width (back / seat plate)	19.75 in.	500mm
Width with side rails (back / seat plate)	21.75 in.	550mm

Adjustments via hand control:

Height	26-45 in.	660-1140mm
Lateral tilt (left/right)	20°	
Trendelenburg	25°	
Reverse Trendelenburg	20°	
Power Kidney Bridge	6 in. Lift	150mm Lift
Back up/down	90°/40°	
Leg up/down	2°/98°	
Return to level (anatomically correct return to horizontal position from articulated positions)		
Top slide/head	7.5 in.	195mm
Top slide/foot	13.5 in.	335mm
Flex/Reflex	40°/90°	
Base braking (lock/unlock)		
Weight of operating table:	827 lbs.	376kg
Max. permissible load:	700 lbs. (320kg) Lift	
	600 lbs. (275kg) Articulation	



Full function, electrohydraulic battery powered surgical table features 21 in. (530mm) longitudinal top slide, 700 lbs. (320kg) lift and 600 lbs. (275kg) articulation capacity. Table is equipped with a removable 15 ft. (457cm) power cord for battery charging. Four (4) conductive casters provide four point braking system with push button operation, including manual override for safety and stability. Hand held, back-lit pendant control includes steel housing design providing extreme durability. Pendant permits control at any point around table perimeter.